

LiceGuard Robi Comb Guide



F 1

F 2

F 3

F 4

Directions for use:

- 1) Insert AA battery in back compartment.(F1)
- 2) Ensure unit is clean before use.
- 3) Turn unit on using side button (F2) - there should be a continuous buzzing noise.
- 4) Do not use on wet hair or if scalp has sores.
- 5) Use unit at a 45 degree angle to the hair roots with white lettered label facing up. (F3)
- 6) Comb through the hair near the scalp where lice live. No need to comb to the end of hair.
- 7) Buzzing will stop each time louse is destroyed. If buzzing does not resume, clean comb teeth using small brush included in kit.(F4)
- 8) Treatment should take 5 to 10 minutes. Turn unit off and clean as directed below.

Cleaning the Comb Teeth:

- A. Remove front guard then remove comb teeth from inside unit .
- B. Brush teeth with small brush to remove any debris.
- C. Wash teeth in a cup of warm, soapy water.
- D. Dry well with paper towel.
- E. Sanitize with antiseptic wipe.
- F. Dry again and put back in Robi Comb.

Video guides available here:

<https://www.lice-guard.com/pages/robicomb-videos-support>